



Perisept/Sanotex Wipes Preparation Guide

- Apply a secondary label, or write the name of the chemical you are using in the section provided on the wipe container.
- When empty, refill with a replacement roll of Sanotex Wipes. ALWAYS use the same chemical marked on the outside of the container.

Recommended Dilution Chart

Number of Wipes	Ounces of Sanotex Solution
40	18 oz.
60	28 oz.

Preparing Sanotex Environmental Surface Roll Wipes



1. Insert a dry roll of wipes into your empty canister.



2. Carefully pour the chemical solution over the center of the wipes roll in the canister. Refer to the recommended dilution chart above.



3. Wait 15 minutes for solution to saturate wipes.



4. From the center of the roll, thread the wipes through the dispensing port approximately 1 inch.

5. To dispense, open the hinged lid and pull wipes through the opening at an angle. Close lid when not in use to keep wipes moist.



Sanotex Environmental Surface Roll Wipes, half-fold, 10"x12"
#85347 (40/roll, 48 rolls/cs) or #85337 (60/roll, 30 rolls/cs)



Sanotex II Canister w/Lid
#PRCN0004 (5/cs)